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Norm: EN-ISO 111 99-1

Please read these safety instructions carefully before use, and keep them for future question.

1. Safety indications

♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.

♥ Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature !!).

♥ Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).

♥ Check carefully all parts of this item (screws, tubes, tips, grips)..

♥ Check regularly if all screws are tightened.

♥ Use only as a commode chair and in proper way : don't stand up on the chair.

♥ ALWAYS observe the weight limit on the labelling of your product.

♥ No modification must be made on this device. Use only original parts to

replace the broken or defect one and let this be done by the technical staff.
♥After unfolding or assembling the chair, make sure that it is securely locked in OPEN position and level to the ground before using. Some chairs are already mounted.
♥DO NOT hang anything on the Chair.
♥ALWAYS test to see that the chair and attachments are properly and securely locked in place before using.
♥In case you didn't use your chair for a long time, let it checked by the technical staff, before using.
♥Do not store outdoor, the chair must be used at home.
♥Do not install or remove the pan while the patient is seating on the chair! High risk of sex injuries.
♥Be carefull when removing the armrests to avoid trapping your fingers.
♥Check the chair before using, in case of doubt ask your physician or provider.
♥Use only the chair on a flat and solid ground.
♥It is forbidden to seat on the armrests.
♥Do not lean out of the chair to avoid losing balance.

2. Installation

Only specialists can adjust the product and teach you how to use it. Ask the technical staff how to use this product and make sure that you understood clearly their explanation.
In case of doubt, don't hesitate to ask again.

There are 2 different ways to install/remove the pan :

1- from the left or right side under the seat, insert the pan into the metallic support.

2- lift up the seatplate and remove the pan from the top.

Close the pan with the lid before transporting it.

The seatplate can be easily removed for cleaning or for replacement as it is only connected to the frame with 2 clips.

By adding the cover on the hole, the commode chair can be used as a regular chair.

3. Cleaning

The chair has to be cleaned regularly and after each use with a mild cleaning agent and a soft cloth.

4. Materials

The product is made up of steel tubing covered with epoxy painting. Upholsteries are in PVC. The chair is delivered with a pan. Some models are equipped with swinging armrests.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

This product has a 2 years warranty, form the date of purchase, against any manufacturing defect.

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