Please read these notes carefully before use. Make sure to follow the instructions and keep them in a safe place ! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

1. Safety indications

- the product must be used in normal conditions and inside home. Do no stock it outside. Do not warm the pan more than 100°C (212°F)
- to avoid any genital wound, do not take out or install the pan when the patient is sitting on the toilet chair.

2. Installation

The pan has to be put under the seat. It can be installed from the right or the left side, or from the top. The seat is fixed to the frame thanks to 2 clips allowing the user to lift it up in order to take out the pan. It is possible to remove the plastic seat with a hole completely, to replace it or to clean it. Put "the cork" in the hole to use the chair as a normal chair.

3. Cleaning

The toilet chair must be cleaned with non-abrasive detergents and dried with a rag after cleaning. Beware: urine is very aggressive and quickly damages the steel tubes of the frame. It is therefore necessary to clean the chair daily.

4. Materials

The toilet chair is made up of a steel frame, a removable seat with a hole, a "cork", and a plastic pan. PVC outside and foam inside.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

Weight: 5.6 kg
Maximal weight (user): 130 kg

*This product has a 2 years warranty, from the date of purchase, against any manufacturing defect.
This product is biocompatible.*

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