ALOVA- Matratze (Matratzenauflage)

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ALOVA (overlay) mattress

TIPS

"The best way to prevent pressure ulcers in the lying down position is to change the contact points of the body on the support on average every three hours to limit compressing vessels for too long around bony prominences and to allow tissues to be irrigated".

INDICATIONS

- Medium to high risk of pressure ulcers evaluated according to the accepted clinical scales and professional judgement.
- Established pressure ulcers: follow medical recommendations.

FITTING INSTRUCTIONS

• Position the three sections of the multiple support (overlay) mattress according to the diagram found on the label sewn on the cover; the pictograms on each of the sections mean their position can be easily marked:

• **"Medium"** support for the head
letter **"m"** identified on the support section.

• **"Firm"** support for central part (pelvis)
letter **"f"** identified on the support section.

• **"Supple"** support for foot section (heels)
letter **"s"** identified on the support section.

CLEANING / DISINFECTION

• ALOVA (overlay) mattress
If necessary, use vaporisation to apply a disinfecting detergent or a surface disinfectant which does not soak the material. Let it dry before covering again.

• Promust PU and Promust PU HD Protective covering

Obligatory CE marking following the directive 93/42/CEE for products claiming use on surfaces of Medical Devices.

SAFETY

- Fit the overlay mattress sections on to a single mattress in good condition.
- Limit it to users weighing between 20 and 120 kg.
- VMA120 : 150 Kg.
- Non edible product, do not ingest.

Precautions for protective coverings

- Systematically protect the (underlay) mattress by means of a protective covering which is always delivered dry: if the foam has repeated contact with fluid, its properties will be modified regarding supporting the body uniformly and constantly as well as helping to lower pressure values.
- Request a new fire classification from an official organisation in the event of using a different protective covering from the one supplied by ASKLÉ.
- Prohibit scouring, stripping or dissolving products and sharp objects in direct contact with the protective covering.
- Make sure you regularly inspect the normal wear on the protective covering and replace it if it is damaged (holes, cracks, changes in the surface, transparent when looked at in the light).
- The use of a single protective covering does not totally control the risk of nosocomial infection: fluids and micro-organisms penetrate the support materials via seams, fastenings or the protective covering itself if it is worn, representing a danger for fragile immunitary users. Within the context of multiple patient use, ASKLÉ SANTÉ recommends the use of Dual Membrane double protective covering +elastic cover to avoid cross contamination risks.

Precautions for materials

- Avoid prolonged damp contact with the foam.
- Carry out regular visual checks on the condition of the foam: the ALOVA material visibly setting or a non uniform and very slow return are all signs of ageing which compromise the support properties which help to prevent pressure ulcers.

STORAGE

- ALOVA supports should preferably be stored flat, out of direct sunlight and excessive damp.

DISPOSAL

- Do not burn or dump the support and its protective covering outside areas intended for this purpose.

WARRANTY

The ALOVA (overlay) mattress is guaranteed for 3 years. This guarantee starts on the date of purchasing the product from your distributor.

In the event of single patient usage, the guarantee covers the support and its protective covering. In the event of multiple patient usage, the length of the guarantee for the protective covering is reduced to 1 year.

This guarantee covers conformity faults which exist when delivered and which appear under ASKLÉ SANTÉ recommended usage conditions. This guarantee does not cover normal wear on the product and its protective covering and does not substitute legal guarantees.

Please contact **YOUR DISTRIBUTOR** and show him your faulty product. They will carry out the necessary procedures with our company in order to carry out repairs or make an exchange.