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Ref. 221000, 221100

Please read these notes carefully before use. Make sure to follow the instructions and keep them in a safe place ! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

1. Safety indications

- You should ALWAYS consult with your physician or therapist to determine proper adjustment and use.
- If walker is exposed to extreme temperature (above 100°F or below 32°F), high humidity and/or becomes wet, prior to use, ensure handgrips DO NOT twist on side frame otherwise damage or injury may occur. Unit should be dry, and handgrips tight.
- Check to make sure rubber tips and/or plastic anti-glide tips are not ripped, worn or missing. Replace before using walker. DO NOT use rubber tips on rear leg extensions when using front wheel or brake options.
- All leg extensions, must be adjusted so the walker is level. When using wheeled attachments, extensions MUST be adjusted so side frames are level.
- Rear extension legs with rubber tips or plastic anti-glide tips MUST be in contact with floor surface at all times.
- The leg extensions should be adjusted so the walker is level. If an even height cannot be achieved, adjust the leg extensions so that the rear of the walker is no more than one inch lower than the front.
- Anti-glide tips are used on rear leg extensions ONLY to allow walker to roll easily over indoor carpeted areas without the need to lift the walker off the ground. Anti-glide tips are not recommended for use on rough surfaces (concrete, gravel, etc).
- Anti-glide tips on rear leg extensions should only be used with front wheel attachments. DO NOT use anti-glide tips on all four leg extensions at once. DO NOT use anti-glide tips and rubber tips at the same time.
- When using anti-glide tips on walker, short steps should be taken to assure weight is distributed evenly and directly over walker legs. These precautions will prevent uncontrollable movement of walker.
- ALWAYS use caution when using the walker, especially on wet or icy surfaces.
- ALWAYS observe the weight limit on the labeling of your walker. Check that all labels are present and legible. Replace if necessary.
- After unfolding or assembling walker, ensure walker is securely locked in OPEN position and level to the ground before using.
- DO NOT hang anything on the front of the Walker. This may depress the release mechanisms and prevent walker from locking into place when fully opened.
- DO NOT hang anything on either the left or right side frames (area that includes dual-release paddles) or the left and right slide tubes (area that includes palm release button). This will cause side frames or slide tubes to bend downward and prevent walker from locking into place when fully opened. This may also cause walker to tip, resulting in injury or damage.
- ALWAYS test to see that the walker and attachments are properly and securely locked in place before using.

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2. Installation

Before using the walking frame, the height has to be adjusted to the user's size. Push the button of each leg and adjust the height: push the tube inside to reduce the height or pull it to increase the height. When adjusted, release the push buttons to fix the height. Both sides must have the same height, and the pushbutton has to come out of both sides.

To move: put the frame forward, lean on it, and move your feet forward.

3. Cleaning

The walking frame has to be cleaned regularly with a dry and clean rag. The handles have to be cleaned with a non-abrasive cream cleaner to avoid dirtiness. The rubber tips have to be changed regularly.

4. Materials

The walking frame is made out of anodised aluminium tubes.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

Weight: 1,9 kg
Maximal weight supported: 100 kg.
Width: 651 mm
Height: 885 - 985 mm
Depth: 470 mm.

*This product has a 2 years warranty, from the date of purchase, against any manufacturing defect.
This product is biocompatible.*

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