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243830 DELTA ROLLATOR

DO NOT install this equipment without first reading and understanding this instruction sheet. If you are unable to understand this instructions, contact a healthcare professional, dealer or technical personnel if applicable before attempting to install this equipment otherwise, injury or damage may occur. Always check Delta before using. The Delta helps people to move safely.

Installing the Delta

1. Remove the plastic tap from the top of the Delta.
2. Place the two handles with brake in each tube (left and right).
3. Adjust the handles to the proper height, Arm must be 45° position to ensure a good confort.
4. Inspect the tub bar for stability by putting a reasonable amount of weight onto the grip and then try to move it from side to side.
5. To stop the Delta, use the 2 brakes by pushing down the plastic handles under the anatomic handle.

6. If any doubt on how using the Delta contact your supplier. If Delta looks instable, DO NOT USE!

CARE AND MAINTENANCE

1. Use mild soap and water to clean.
2. An antibacterial cleaner is recommended for sterilization.
3. Use a silicone lubricant for the threads of the adjustment knob to inhibit rust.
4. Periodically inspect the tub bar for wear, damage, etc. DO NOT use if any damage exists.

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