
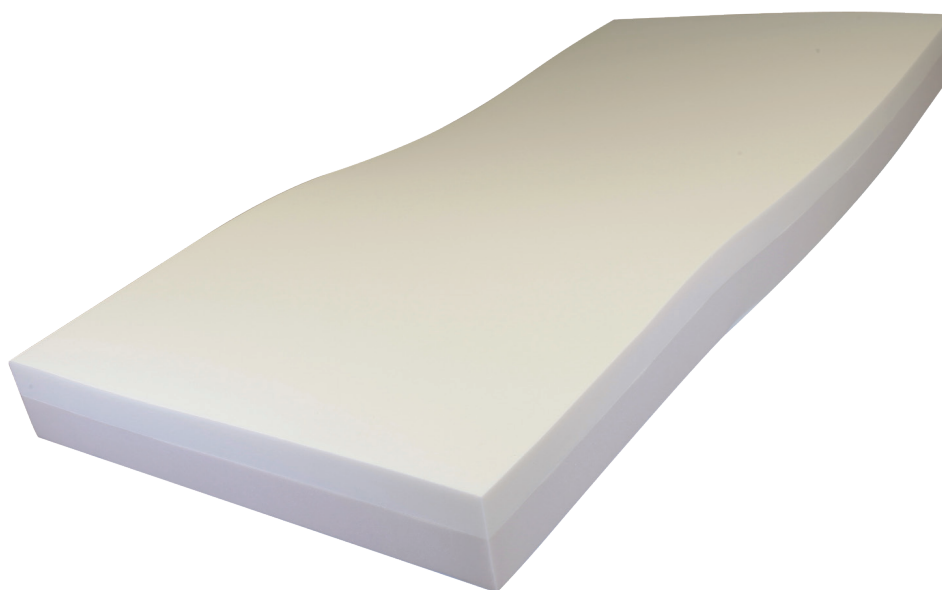
Model	Designation	E	A	B	C	D			
410064	Topi combi 70 cm	14 cm	8 cm	6 cm	198 cm	70 cm	86 kg/m ³ + 21 kg/m ³	M4, cover M1	9.6 Kg
410061	Topi combi (visco + polyether)	14 cm	8 cm	6 cm	198 cm	88 cm	86 kg/m ³ + 21 kg/m ³	M4, cover M1	12 Kg
410060	Maxi (visco + HR foam)	14 cm	8 cm	6 cm	198 cm	88 cm	86 kg/m ³ + 38 kg /m ³	M4, cover M1	15 Kg
410067	Maxi 120 cm	14 cm	8 cm	6 cm	198 cm	120 cm	86 kg/m ³ + 38 kg /m ³	M4, cover M1	20.6 Kg
410068	Maxi 140 cm	14 cm	8 cm	6 cm	198 cm	140 cm	86 kg/m ³ + 38 kg /m ³	M4, cover M1	23 kg
410069	Maxi Tri	16 cm	10 cm	6 cm	198 cm	88 cm	86 kg/m ³ + 38 kg /m ³	M4, cover M1	16.5 kg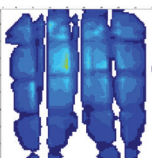

Sleepy		Fatigue Test		Pressure Distribution		Skin Microclimate	
				Heel (mmHg)	Pelvis (mmHg)	Heat Transfer rate (Wm-2K-1)	Water Vapour Transfer rate (gm-2day-1)
		4%		92 +/-4	75 +/-7	15.2 +/-0.05	477 +/-2

Maxi Tri		Fatigue Test		Pressure Distribution		Skin Microclimate	
				Heel (mmHg)	Pelvis (mmHg)	Heat Transfer rate (Wm-2K-1)	Water Vapour Transfer rate (gm-2day-1)
		4%		77 +/-4	75 +/-7	16.5 +/-0.05	477 +/-2

Topi Combi		Fatigue Test		Pressure Distribution		Skin Microclimate	
				Heel (mmHg)	Pelvis (mmHg)	Heat Transfer rate (Wm-2K-1)	Water Vapour Transfer rate (gm-2day-1)
		4%		77 +/-4	79 +/-7	14.5 +/-0.05	560 +/-2

Maxi		Fatigue Test		Pressure Distribution		Skin Microclimate	
				Heel (mmHg)	Pelvis (mmHg)	Heat Transfer rate (Wm-2K-1)	Water Vapour Transfer rate (gm-2day-1)
		4%		62 +/-4	70 +/-7	15 +/-0.05	570 +/-2

Please read these safety instructions carefully before use, and keep them for future question.

1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature !!), store in a dry room in a flat or rolled position to prevent deforming, stretching and tearing of the mattress.
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F). Indoor use only.
- ♥ Store away from sharp objects and pointed edges to prevent tears and punctures.
- ♥ Use in a flat unfolded position.
- ♥ Transport rolled or flat to avoid tears and punctures.
- ♥ No modification must be made on this device.
- ♥ ALWAYS test to see that the mattress is properly in place before using.
- ♥ In case you didn't use your mattress for a long time, let it checked by the technical staff, before using.
- ♥ ALWAYS observe the weight limit on the labelling of your mattress.
- ♥ Follow the advice of the professional who prescribed this medical device, respect duration and frequency of use.
- ♥ If your skin develops redness, discontinue the use of the mattress immediately and see your doctor or therapist.
- ♥ In case of discomfort consult your professional.
- ♥ Do not place in or drop into water or other liquids.

2. Indication

It is easier to prevent pressure ulcers than to treat them. Bedsores can be avoided by changing frequently lying or seating positions to avoid stress on the skin and to minimize the risk of pressure on vulnerable bony areas. Other strategies include skin care, regular skin inspections and good nutrition.

3. Description and Fitting instructions

Beware : always place a bed sheet between patient and mattress.

The static viscoelastic mattress helps preventing and treating bedsores by improving the blood circulation and the risk of pressure on vulnerable bony areas.

When delivered rolled, first remove the mattress from its bag. After 24 hours the mattress will recover back to its correct depth.

The mattress is delivered with a Techmaflex® cover.


The mattress consists in a Polyether (Topi Combi®) or bultex® High Resiliency (Maxi® and Maxi Tri®) base and a memory foam overlay of a very high density (86 kg/m³).

The etiquette sewn on the cover indicates the base of the mattress. Place the mattress on the bed with the memory foam on the top.

4. Washing instructions

To clean the cover:

The cover is machine washable at 90°C. It is fire resistant classified M1, waterproof and vapour permeable. Dry flat and away from any source of heat.

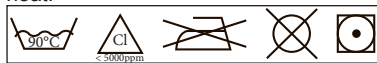To clean the mattress:

Remove the cover, then wipe lightly with a damp cloth. Do not use soap and do not submerge in water. After cleaning make sure to wipe dry and air dry the mattress before using. Remove the cover, then wipe lightly with a damp cloth. Look for tears, scratches and punctures in the cushion before reinstalling the cover. In case of problem contact your technical staff.

5. Composition

The mattress is made from various Polyether (Topi Combi®) and Polyurethane (High Resiliency 38kg/m³ or viscoelastic 86kg/m³) foams (Maxi® et Maxi Tri®) having various densities and hardnesses.

The cover is made from polyurethane (66%) and polyester (34% PES) for a total weight of 140g/m².

6. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

7. Technical specifications

See tabel.

This product has a 3 year warranty, from the date of purchase by the patient, against any manufacturing defect.

This product is biocompatible and CE certified.

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