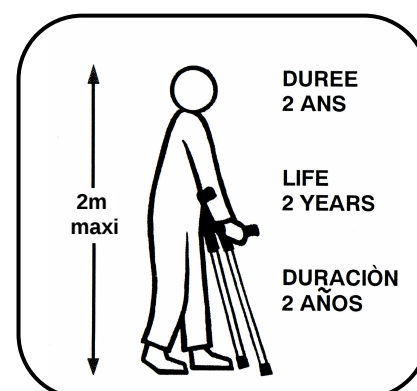
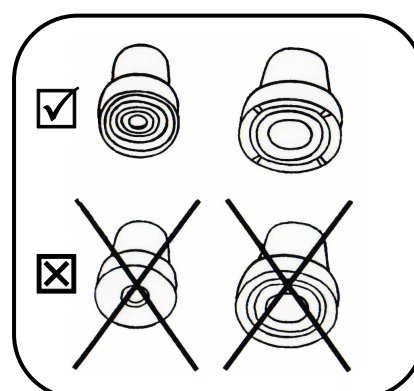
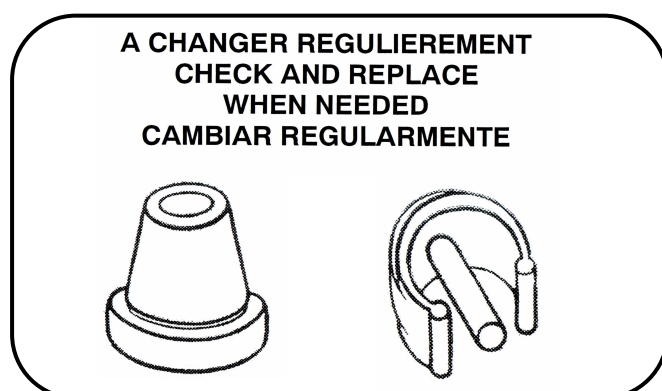
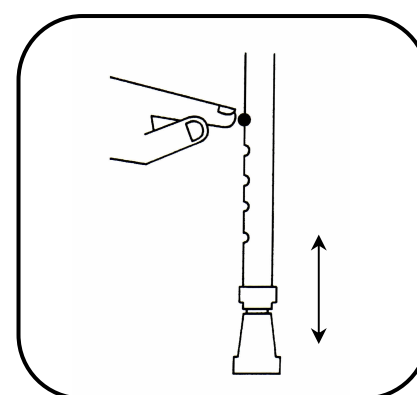
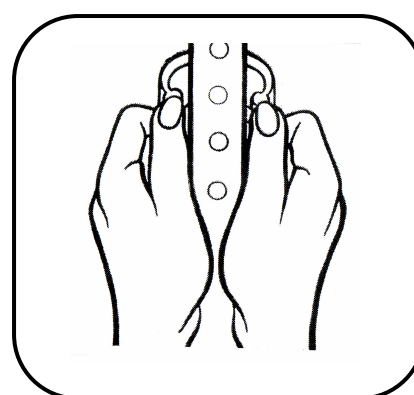
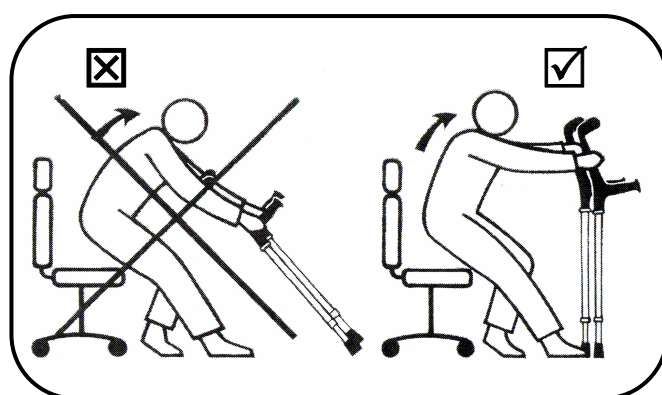
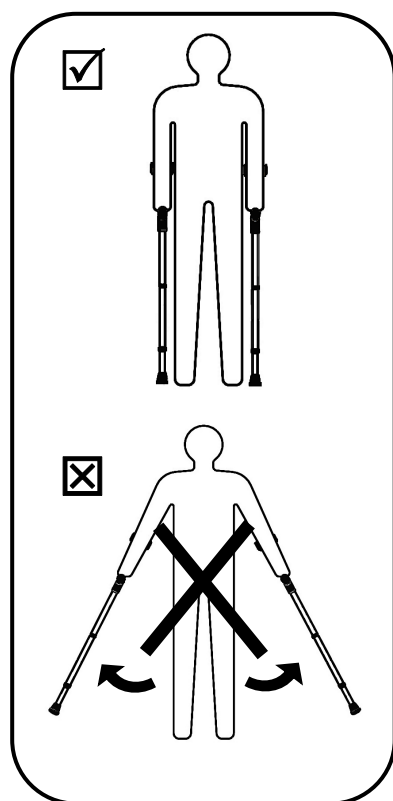
Fr

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En

Congratulation for your acquisition of HMS-VILGO crutches.

These crutches have been especially made and manufactured for a medical / therapeutical use of walking aid.

Height adjustment:

The grip must be adjusted at the same height than the hip (head of femur)

- Height adjustment by clip: Make sure that the clip is adjusted around the tube and that it goes through the tube.
- Height adjustment by ratchet: Make sure that the ratchet goes completely out of the tube. Make sure that the crutch is completely blocked in pressing down progressively on the grip.

Warning and precautions of use:

- Never overpass the maximum weight indicated on the crutch stick.
- Exclusively use the crutches on a flat and solid ground.
- Using crutches on stairs is not recommended.
- Adapt your walking speed to the environment – Be very prudent on humid, cold or slippery ground.
- Never use the crutches for other use than walking aid.
- Never modify the product, that could have a negative influence on the resistance or stability of the disposal.
- Change the ferrules regularly and as soon as it is necessary.

Warranty:

Your crutch has two years warranty against all fabrication vices. Wear and accidental damages are not covered by this warranty.

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