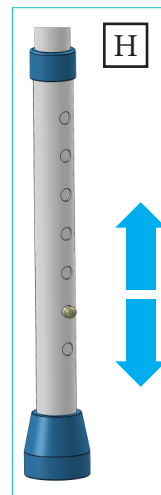
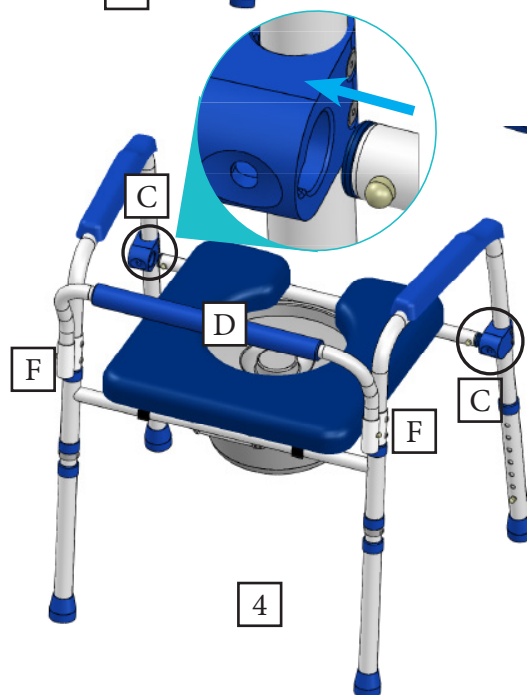
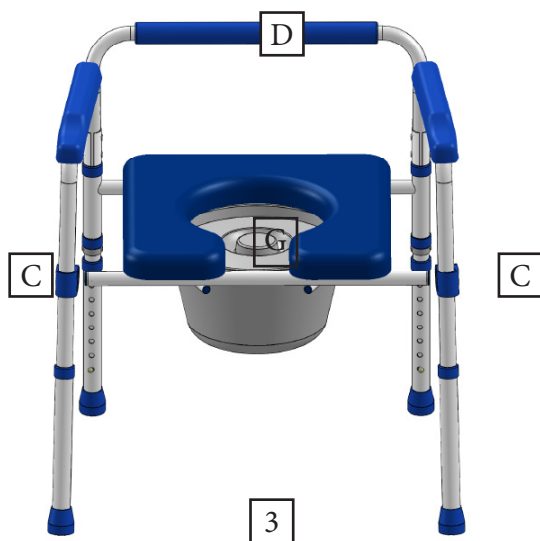
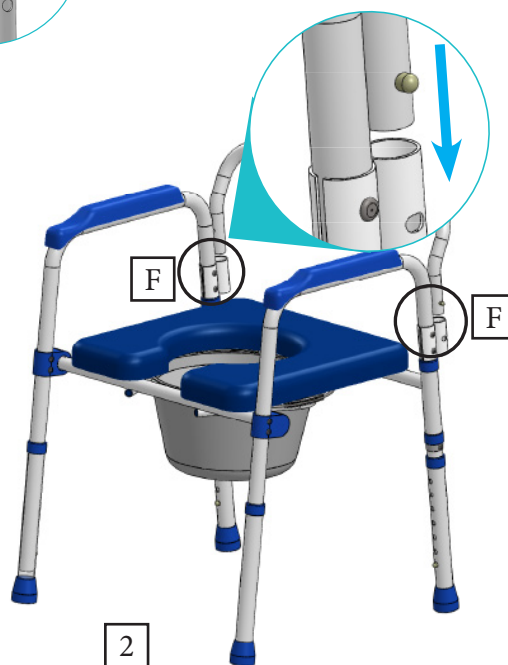
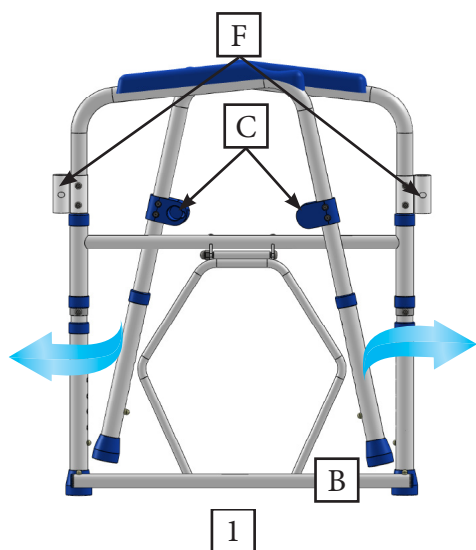
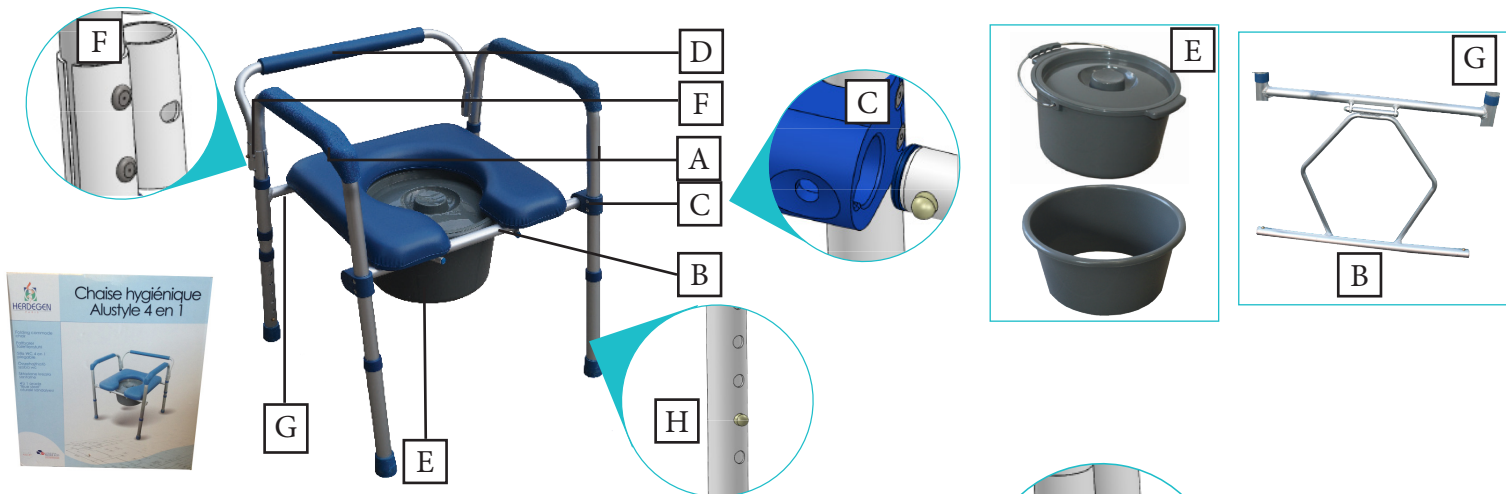
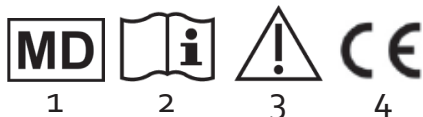
Model N°	A & D	B	C	E	F	G	H	Gewicht	Max. Gewicht
380004 Alustyle	63.5/78.5 cm	46 cm	54 cm	45/60 cm	18.5 cm	40 cm	51-54 cm	3.2 kg	120kg
Taille de l'ouverture/Dimension of the hole/Maße der Aussparung: $\varnothing = 20.5$ cm									

1 2 3 4

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EN- Please read these safety instructions carefully before use, and keep them for future question.

1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- ♥ Check carefully all parts of this item (screws, tubes, tips, grips)..
- ♥ Check regularly if all screws are tightened.
- ♥ Use only as a commode chair and in proper way : don't stand up on the chair.
- ♥ ALWAYS observe the weight limit on the labelling of your product.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After unfolding or assembling the chair, make sure that it is securely locked in OPEN position and level to the ground before using. Some chairs are already mounted.
- ♥ DO NOT hang anything on the Chair.
- ♥ ALWAYS test to see that the chair and attachments are properly and securely locked in place before using.
- ♥ In case you didn't use your chair for a long time, let it checked by the technical staff, before using.
- ♥ Do not store outdoor, the chair must be used at home.
- ♥ Do not install or remove the pan while the patient is seating on the chair! High risk of sex injuries.
- ♥ Be careful when removing the armrests to avoid trapping your fingers.
- ♥ Check the chair before using, in case of doubt ask your physician or provider.
- ♥ Use only the chair on a flat and solid ground.
- ♥ It is forbidden to seat on the armrests.
- ♥ Do not lean out of the chair to avoid losing balance.
- ♥ Any serious incident that has occurred in relation to the device shall

be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

2. Intended use, indications and contraindications

Intended use:

Commode chairs can be used at home or in nursing homes to assist a person who has difficulty getting to the bathroom by bringing the toilet into the bedroom. Standard, foldable or on castors, they shall be use only indoor.

Indications

Used in the home when the patient is too debilitated to reach the bathroom. Lack of trunk control - Loss of balance - Paralysis - Walking problems - Overall weakness - Stress incontinence - Fecal incontinence - Chronic diarrhoea

Contraindications

Don't use in case of:

- Severe cognitive disorder
- Severe loss of balance
- Sitting impairment

Only specialists can adjust the product and teach you how to use it. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again.

Assembly:

1. Remove commode from packaging and check for cracks and damages.
2. Unfold commode's sides [A] and stand upright with sides perpendicular to the backframe [D] see drawing [1].
3. Attach backrest tube [D] by depressing push buttons and sliding brace into side frame socket [F] located on each side at the back of the chair see drawing [2]. Beware risk of injury! Make sure that push button "snap" correctly through holes and that the backrest is connected to the frame.
4. Lift up front tube [B] of the bracket [G] see drawing [2] and attach it by depressing push buttons and sliding brace into side frame sockets [C] located on each side of the chair see drawings [3] and [4]. Beware risk of injury! Make sure that push button "snap" correctly through holes and that the chair is stable.
5. Install commode bucket and anti splash guard [E] into bracket [G] and clip seatplate onto back of frame.
6. To unfold, depress push button on back support and remove. Depress push button on front frame and remove the front tube.
7. To remove the pan, lift up the seatplate and remove the pan from the top. Close the pan with the lid before transporting it.
8. Lift up the seatplate to remove it.

Leg adjustment see drawing [H]:

Depress the push button in the frame and adjust the leg extension to the required height adjustment hole. Make sure that the push button is fully engaged through the extension leg hole. Check the stability before use. Adjust all legs at the same level.

Rental: this chair can be rented under the condition that it was correctly cleaned. If necessary, replace the lid and the seatplate before the next rental. This user's manual belongs to that product and must be delivered to each new user.

3. Cleaning

The chair has to be cleaned regularly and after each use with a mild cleaning agent and a wet cloth.

4. Materials

The product is made up of aluminium tubing covered with epoxy painting. Upolsteries are in PVC. The chair is delivered with a pan.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

Please see above information on page 1. Duration of use: 2 years.

This product has a 2 years warranty, from the date of purchase, against any manufacturing defect. This product is biocompatible. Test : EN-ISO 111 99-1.

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