NO.1 CHOICE OF HOSPITALS AND MOTHERS

# Symphony®

BREASTPUMP      TIRALATTE  
BRUSTPUMPE      BORSTKOLFF  
TIRE-LAIT

**EN** INSTRUCTIONS FOR USE

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
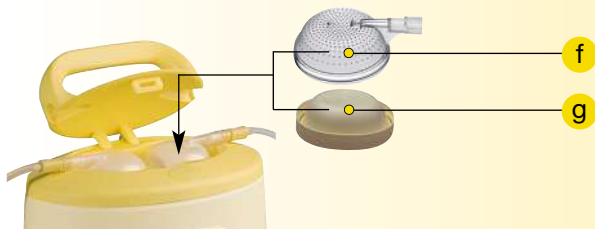
<b>EN</b>	Instructions for use	2
<b>DE</b>	Gebrauchsanweisung	18
<b>FR</b>	Mode d'emploi	34
<b>IT</b>	Istruzioni per l'uso	50
<b>NL</b>	Gebruiksaanwijzing	66
	EMC	84

# I Symphony® Breastpump

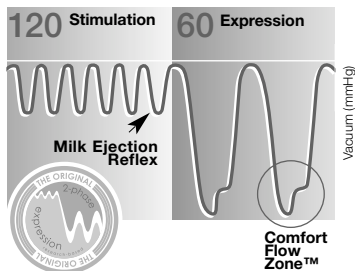Mother's milk is the best nutrition for babies. Symphony gives you and your baby optimal support during the breastfeeding period. The attractive breastpump for hospitals and home rental is especially suitable for long-term and frequent pumping and encourages milk production in the long term, too.

**Symphony** has **2-Phase Expression®**, the only research based breastpump technology to mimic a baby's natural nursing rhythm.

## **Stimulation:**

Fast pumping rhythm to start your milk flowing.

## **Expression:**

Slower pumping rhythm to express your milk gently and efficiently.

## **Comfort Flow Zone™:**

For maximum comfort and efficiency – gentle to the breast tissue.

**Proven to feel most like your baby and to reduce your pumping time.**

# II Indications For Use

The Symphony Breastpump is a powered breastpump:

- To be used by lactating women to express and collect milk from their breast.
- To initiate and maintain a milk supply as well as collect breastmilk at work or during other absences from the baby.
- To initiate and maintain a milk supply if direct breastfeeding is not possible.
- To relieve engorgement.
- For use in hospitals or home rental.

## Important Safeguards

*When using electrical devices, basic safety precautions should always be followed, including the following:*

*Read all instructions before use.*

- *Use the product only for its intended use as described in this instruction manual.*
- *Only use the power cord that comes with the Symphony Breastpump.*
- *Make sure the voltage of the device is compatible with the power source.*
- *Plug the power cord into the Symphony Breastpump first and then into wall socket.*
- *Always unplug electrical devices immediately after using.*
- *The product should never be left unattended when plugged into a power source.*
- *Separation from the mains is only assured through the disconnection of the power cord from the wall socket outlet.*
- *Keep the power cord away from heated surfaces.*
- *Never operate an electrical device if it has a damaged power cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.*
- *Keep appliance dry and do not place in direct sunlight.*
- *Do not get the product wet.*
- *Do not use while bathing or showering.*
- *Do not place in or drop pump unit into water or other liquid and do not shower the diaphragm area.*
- *Do not spray or pour liquid directly onto the pump.*
- *Do not store or use this product where it can fall or be pulled into a tub or sink.*
- *Do not reach for any electrical device if it has fallen into water. Unplug immediately.*
- *Use Medela original accessories only.*
- *Check pump kit components for wear or damage.*
- *The pump kit is a personal care item and is not to be shared between mothers without proper sterilization to prevent the risk of cross-contamination.*
- *Pump only with the motor in an upright position.*
- *Do not use while sleeping or overly drowsy.*
- *Supervision is necessary when this product is used in the vicinity of children.*
- *If problems or pain occur, consult your lactation consultant or physician.*
- *Equipment not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide.*
- *Portable and mobile radio frequency communications equipment can affect the Symphony Breastpump.*
- *The Symphony Breastpump needs special precautions regarding EMC (Electromagnetic compatibility) and needs to be installed and put into service according to the EMC information provided in these instructions for use.*
- *There are no user-serviceable parts inside the Symphony. Do not attempt to repair the Symphony Breastpump yourself.*

SAVE THIS INSTRUCTION MANUAL

## IV Assembly Instructions

1

**b + c**

Snap membrane onto valve until membrane lies completely flat.

2

+

**b c**

Push assembled valve and membrane onto breastshield. Ensure sidewise position of valve as indicated on picture.

3

+

**d**

Screw bottle into breastshield.

4

**a b c d**

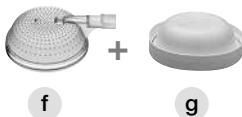
+

**e**


Insert adapter end of tubing into opening in breastshield.

Assemble protective membrane onto membrane cap.

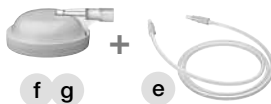Push adapter end of tubing into opening on membrane cap.

To unlock lid, keep pressing the oval release button on top of the Symphony and lift up the handle to get access to breastpump diaphragm.

Push membrane tightly into place on breastpump diaphragm. To ensure proper fit, be sure to center the membrane caps' tube port directly over the crevice on the pump housing. Make sure that the membrane clicks into place and is flush with surrounding surface.

**Close the lid.**

## V Operation

### Electrical

Power the Symphony (or charge its battery if included) by connecting the socket in the back of the pump and the power cord to a standard electrical outlet.

**Battery** optional (built-in rechargeable battery)

Charge Symphony pump batteries for 12 hours upon receipt.

Optimal battery life is achieved when:

- Batteries are not fully drained on a regular basis
- Pump is allowed to charge when not in use

When not in use for a lengthy period, recharge batteries for 12 hours every two months.

Pumping time (full battery) = approx. 60 minutes

Charging time (empty battery) = 12 hours

Pump may be used while charging.

## VI Pumping

*Be comfortably seated and relax. Use a footrest, when pumping or when feeding your baby. Elevating your feet puts you into proper sitting position to avoid stress on the lower back and perineum.*

Center your breastshield over your nipple.

- Do not tilt or overfill bottles when pumping.
- Take care not to kink tubing while pumping.

**2**

Switch on the breastpump by pressing the On/Off button. The pump will begin pumping in the **Stimulation Phase**.

Adjust the vacuum during pumping to your comfort level by turning the one-knob control clockwise (increase) or counter-clockwise (decrease).

**3**

Once your milk starts to flow or you sense milk ejection, press the let-down button to switch to the **Expression Phase**.

If you do not press the let-down button, the pump will automatically switch to the Expression Phase after two minutes.

Set your **Maximum Comfort Vacuum™** which is the highest vacuum you can use and still feel comfortable (this is different for every mother). To find your Maximum Comfort Vacuum increase vacuum until pumping feels slightly uncomfortable (not painful), then decrease slightly.

**Research has shown that you will pump more efficiently – meaning get more milk in less time – when pumping at Maximum Comfort Vacuum™ in the Expression Phase.**

Once finished pumping, switch off the breastpump, unplug it, and disassemble the pump kit items. Close the bottle and store it for later use.

*Use bottle stand to prevent tip over.*

## VII Symphony® Program Cards

There are two versions of the Symphony Program Card:

- The Original Symphony Program Card (phase-out in the course of 2007)
- The new Symphony Program Card 2.0 (available as of mid 2007)

The following information is an explanation for either card.

### Original Symphony Program Card

#### LCD Display

When powered off and connected to wall socket, the LCD Display will indicate «Mains».

When batteries are used to power the Symphony, the LCD Display will remain blank until powered on.

#### Stimulation Phase

The initial vacuum setting will be indicated by **one drop** on the LCD Display. Stimulation Phase is limited to two minutes.

#### Switch to Expression Phase

The pump will switch to Expression Phase by one of two ways:

- 1) automatically, after the two minute Stimulation Phase is complete, or
- 2) manually, by pressing the let-down button.

The vacuum setting will be indicated by bars on the LCD Display.

#### Vacuum Setting

When switching from Stimulation to Expression, the pump automatically increases the vacuum setting by adding three bars to the last Stimulation setting.

This vacuum level can be increased and decreased by using the one-knob control.

## Symphony Program Card 2.0

The Symphony Program Card 2.0 is the second generation program card. It is programmed to operate 2-Phase Expression similar to the Original Symphony Program Card. Additionally, there are some new features available:

### LCD Display indicates Program Card Name

The Symphony Program Card 2.0 displays the name of the card «Symphony 2.0» on the LCD Display when the pump is powered off and connected to wall socket.

When batteries are used to power the Symphony, the LCD Display will remain blank until powered on.

### Stimulation Phase starts with Minimum Vacuum Setting

The initial vacuum setting is visually indicated with **three drops** on the LCD Display. Stimulation Phase is limited to two minutes.

### Switch to Expression Phase

The pump will switch to Expression Phase by one of two ways:

- 1) automatically, after the two minute Stimulation Phase is complete, or
- 2) manually, by pressing the let-down button.

### Ramp-up Vacuum Transition from Stimulation to Expression Phase

When switching from Stimulation to Expression Phase, the Symphony Program Card 2.0 increases the vacuum setting by adding three bars to the last Stimulation setting in a stair-stepped fashion. The three additional bars and increased vacuum setting occur gradually over a 4-second period.

The ramp-up transition eliminates potential discomfort of an immediate vacuum setting increase.

### **Switching back and forth between Stimulation and Expression Phase by using the Let-Down Button**

With the Symphony Program Card 2.0, the let-down button can also be used to switch back from Expression to Stimulation Phase (for more information on this feature see page 11 «Addressing Special Needs»). On the LCD Display the vacuum setting will be indicated by drops in the Stimulation Phase and by bars in the Expression Phase.

### **Program Remembers Last Stimulation Vacuum Setting**

When switching back to Stimulation Phase, the Symphony Program Card 2.0 remembers the last Stimulation Phase vacuum setting prior to switching to Expression Phase and returns to this setting in Stimulation Phase.

The Symphony Program Card 2.0 includes a feature that allows mothers to switch back to the Stimulation Phase by pressing the let-down button. Some mothers find this feature beneficial in addressing the following special needs:

### **No milk ejection occurs in the first two minutes of the Stimulation Phase**

If your milk ejection has not occurred in the first two minutes of the Stimulation Phase and the pump automatically switches to the Expression Phase, press the let-down button once to return to the Stimulation Phase. If you have not achieved a milk ejection after two consecutive Stimulation Phases, take a break, relax and try again in 15-30 minutes.

### **Engorgement**

If, due to engorgement, you are experiencing difficulties breastfeeding or expressing milk, careful use of the Stimulation Phase may help to provide some relief. If your milk ejection does not occur in the first two minutes of the Stimulation Phase and the pump automatically switches to the Expression Phase, press the let-down button to return to the Stimulation Phase. Repeat this process until your milk ejection occurs. After milk ejection you may find it beneficial to remain in the Stimulation Phase until enough milk has been expressed so that the nipple and areola area are not as hard and engorged. After initially draining the breast to relieve the discomfort, follow up with breastfeeding or pumping in the Expression Phase to adequately drain the breast. Although Medela strongly encourages pumping in the Expression Phase after milk ejection at Maximum Comfort Vacuum for the most effective removal of milk, it may be necessary to address temporary discomfort issues by pumping in the Stimulation Phase.

*If your engorgement condition persists or worsens, consult your physician or lactation consultant.*

### **Sore Nipples**

If, due to sore nipples, you are experiencing discomfort while pumping in the Expression Phase, careful use of the Stimulation Phase may help to provide some relief. You can return to the Stimulation Phase at any time by pressing the let-down button. Although Medela strongly encourages pumping in the Expression Phase after milk ejection at Maximum Comfort Vacuum for the most effective removal of milk, it may be beneficial to address temporary discomfort issues by pumping in the Stimulation Phase.

*If your sore nipple condition persists or worsens, consult your physician or lactation consultant.*

## IX Cleaning and Hygiene

### Important

- Use drinking water quality for any of the following described cleaning procedures.
- Clean parts immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.
- Disassemble all parts prior to cleaning. Especially be sure to remove the yellow valve head from the breastshield and separate the white valve membrane from the valve head.

### Cleaning at home

Follow these cleaning instructions unless you are told otherwise by your healthcare professional.

#### Prior to first use and once daily

- Disassemble, wash and rinse all parts of your pump kit that come in contact with the breast and milk (a+b+c+d+h+i+k).
- Place parts in a saucepan of sufficient capacity.
- Fill with cold water until the parts are well covered. Optionally, a tea spoon of citric acid may be added to avoid a build-up of limescale.
- Heat it up and allow to boil for 5 minutes.
- Remove from stove and drain water off.
- Dry parts with a clean towel.

Alternatively, the disassembled parts can be sanitized in top rack of dishwasher or steam-sterilized.

*Note: Food colorings may discolor components which will not affect performance.*

#### After each use

- Disassemble and handwash all parts of your pump kit that come in contact with the breast and milk (a+b+c+d+h+i+k) in warm, soapy water.
- Rinse in clear water. Dry parts with a clean towel.

### Cleaning in hospitals

The following cleaning instructions are general recommendations and may individually be adjusted based on the hospital's specific directives and cleaning practices.

### Important

It is recommended to provide each mother an own personal pump set and to clean, disinfect and sterilize (e.g. autoclave) all components after each use or, latest each time they are handed over to the next mother. Use by more than one mother without adequate sterilization may pose a health risk (e.g. risk of cross-contamination) and voids the warranty.

### Cleaning

Disassemble and thoroughly wash all parts in warm, soapy water. Rinse in clear water.

### Disinfecting

Disinfect all parts by washing in a thermic washing machine (at approx. 93° C).

### Sterilizing

Prior sterilizing visually check parts if clean and intact. Sterilize all parts by autoclaving at 134°C for 3 (or 18) minutes or at 125°C for 15 – 20 minutes. Do not stack parts or apply external pressure to avoid damages. Alternatively, the parts can be EO-sterilized (Ethylene Oxide).

## **Tubing care**

Inspect the tubing after each pumping session for condensation and/or milk.

### If condensation appears in the tubing

Let the pump run after you have completed pumping with tubings attached for 1–2 minutes or until dry.

### If milk gets into the tubing

If milk accidentally gets into the tubing turn off the pump and unplug from power source. Remove and clean the tubing according to the instructions «Prior to first use and once daily». Shake water droplets out and hang to air-dry. For faster drying, attach the tubing to pump and run for 1–2 minutes or until dry.

## **Hints**

After cleaning, when parts are not in use put in a clean zip lock bag or store in a container with lid. Alternatively, the parts can be wrapped in a clean paper or cloth towel.

## **Hygiene**

- Wash hands thoroughly with soap and water before touching the breast and pump parts.
- Avoid touching the inside of bottles and lids.
- Clean the breast and surrounding area with a damp cloth.
- Do not use soap or alcohol which can dry the skin and cause cracked nipples.

## IX Cleaning and Hygiene

### Breastpump care

#### *Unplug breastpump prior to cleaning*

At home

Daily or when spills occur

In hospitals

Immediately when spills occur

In between each mother and/or on a regular basis, e.g. daily

- Use a damp (not wet) cloth to wipe over the breastpump. Optionally, soapy water or a non-abrasive detergent may be used.
- Do not spray or pour liquid directly onto the pump.
- Do not immerse the pump unit in water or shower the diaphragm area.

## X Storing / Thawing Breastmilk

*Check with your hospital for specific instructions.*

### Breastmilk Storage (For Healthy Term Babies)

	Room Temperature	Refrigerator	Refrigerator Freezer Compartment	Deep Freezer
Freshly expressed breastmilk	<i>Do not leave at room temperature</i>	<i>3 – 5 days at approx. 4° C</i>	<i>6 months at approx. -16° C</i>	<i>12 months at approx. -18° C</i>
Thawed breastmilk (previously frozen)	<i>Do not store</i>	<i>10 hours</i>	<i>Never refreeze</i>	

- *When freezing do not fill bottles/bags more than 3/4 full to allow space for expansion.*
- *Label bottles/bags with the date of pumping.*
- *In order to preserve the components of the breastmilk, thaw in the refrigerator overnight or by holding the bottle/bag under running warm water (max. 37° C).*
- *Gently shake the bottle/bag to blend any fat that has separated.*
- *Do not thaw frozen breastmilk in a microwave oven or in a pan of boiling water (danger of burning).*

## XI Troubleshooting

### If the motor is not running

#### Check

- mains/battery power is present
- the device is switched on
- program card is inserted at the back of the breastpump

### If there is low or no suction

#### Make sure

- the breastshield forms a complete seal around the breast.
- all connections of the kit and breastpump are secure.
- the white valve membrane is laying flat against the yellow valve head. Check that the valve head and membrane are clean and not damaged. See «Assembly Instructions».
- the ends of the tubing are fit securely into the back of the breastshield and into the opening on membrane cap.
- the protective membrane / membrane cap are correctly assembled and are pushed firmly into place on breastpump diaphragm.
- the protective membrane is intact.

***Take care not to kink or pinch tubing.***

### If there is an overflow

#### If milk accidentally gets into the tubing

- Turn off the pump and unplug from power source.
- Use a damp (not wet) cloth to wipe the pump and diaphragm.
- Refer to cleaning instructions «Tubing care and Breastpump care».
- If necessary clean the pump kit.
- When components are completely dry, reassemble breast-pump and kit.

## **XII Warranty**

3-year warranty for the unit from the date of purchase, excluding pump set and accessories. 6 months warranty for the batteries. The manufacturer is not liable for any damage or consequential damage caused by incorrect operation, inappropriate usage or use by unauthorized persons. Routine checks, servicing and battery exchange may only be carried out by Medela authorized personnel.

## **XIII Disposal**

The unit is made of various metal and plastics. Before disposal, the device is to be rendered unusable and it must not be disposed of as unsorted municipal waste in accordance with local regulations. Use your local return and collection system for waste electrical and electronic equipment (incl. batteries). Improper disposal may have harmful effects in the environment and on public health.

## XIV Parts List

### Article numbers

a	Breastshield	810.0131
b	Valve head	810.0475
c	Valve membrane	820.7013
d	Bottle (150 ml)	811.0020
e	Silicone tubing (105 cm)	800.0828
f	Membrane cap	810.0667
g	Protective membrane	820.0094
h	Cap	810.0474
i	Lid	810.7062
k	Disc	810.0473

### **Accessories** (sold separately)

- Sterile One-Day Pump Set for a limited number of uses (max. 24 hours or 8 pumping sessions) 800.0830
- Sterile Disposable bottles (80 ml) for one-time use only 800.0835
- Car adapter 928.1017

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Electromagnetic Compatibility (EMC, IEC 60601-1-2:2001)

IEC 60601-1-2, Table 201

<b>Electromagnetic Emissions</b>		
The Symphony breastpump is intended for use in the electromagnetic environment specified below. The customer or the user of the Symphony breastpump should assure that it is used in such environment.		
<b>Emission Tests</b>	<b>Compliance</b>	<b>Electromagnetic environment - guidance</b>
RF emissions CISPR 11	Group 1	The Symphony breastpump uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The Symphony breastpump is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations / flicker emissions IEC 60000-3-3	Complies	

**Warning** – The use of accessories and cables other than those specified may result in increased emissions or decreased immunity of the Symphony breastpump. The Symphony breastpump should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the Symphony breastpump should be observed to verify normal operation in the configuration in which it will be used. The Symphony breastpump was found to have no essential performance by a risk analysis and thus, was not tested for immunity to electromagnetic disturbances (IEC60601-1-2:2001, 6.8.3.201 i).

Technical specifications • Technische Daten • Caractéristiques techniques • Caratteristiche tecniche • Technische gegevens

vacuum (approx.)  
 -50 .....-250 mmHg  
 -7 .....-33 kPa  
 54 .....120 cpm

100 – 240 V  
 50 / 60 Hz  
 0.5 A

T 800 mA / 5x20 / 250 V

12 V DC Car adapter  
 25 W Autoadapterkabel  
 T 2.5 A Câble de branchement sur allume-cigares  
 Adattatore per automobile  
 Auto-adapter

Type 600.0837 / 600.0838 only  
 ■ 2 x 6 V, 1.2 Ah  
 ■ Type Yuasa NP 1.2-6

900 410 410 mm  
 ↓ ↔ ↗  
 245 215 155 mm

3.5 kg  
 2.9 kg

Please see accompanying papers  
 Begleitpapiere beachten  
 Respecter les consignes des documents d'accompagnement  
 Consultare la documentazione allegata  
 Zie bijgevoegde documenten

Must not be disposed of together with household refuse  
 Darf nicht zusammen mit Hausmüll entsorgt werden  
 Ne doit pas être éliminé avec les ordures ménagères  
 Non deve essere smaltito con i rifiuti domestici  
 Mag niet samen met huisvuil weggegooid worden

Operation  
 Betrieb  
 En Service  
 Utilizzo  
 Gebruik

Transport/Storage  
 Transport/Lagerung  
 Transport/Stockage  
 Transport/Stoccaggio  
 Transport/Opslag

Operation  
 Betrieb  
 En Service  
 Utilizzo  
 Gebruik

Transport/Storage  
 Transport/Lagerung  
 Transport/Stockage  
 Transport/Stoccaggio  
 Transport/Opslag

kPa

Protection class II  
 Type B

FDA (GMP)  
 ISO 9001  
 ISO 13485  
 CE (93/42/EEC)

CE  
 0123

SYMPHONY BREASTPUMP  
 WITH RESPECT TO ELECTRIC SHOCK FIRE  
 AND MECHANICAL HAZARDS ONLY IN  
 ACCORDANCE WITH  
 UL2601-1/CAN/CSA C22.2 No. 601.1 22PL

### **International Sales**

Medela AG, Medical Technology  
Lättichstrasse 4b  
6341 Baar/Switzerland  
Phone +41 (0)41 769 51 51  
Fax +41 (0)41 769 51 00  
e-mail [info@medela.ch](mailto:info@medela.ch)  
**[www.medela.ch](http://www.medela.ch)**

### **France**

Medela France Sarl  
14, rue de la Butte Cordière  
91154 Etampes cedex  
Phone +33 (0)1 69 16 10 30  
Fax +33 (0)1 69 16 10 32  
e-mail [info@medela.fr](mailto:info@medela.fr)  
**[www.medela.fr](http://www.medela.fr)**

### **Germany**

Medela Medizintechnik  
GmbH & Co. Handels KG  
Postfach 1148  
85378 Eching  
Phone +49 (0)89 31 97 59-0  
Fax +49 (0)89 31 97 59 99  
e-mail [info@medela.de](mailto:info@medela.de)  
**[www.medela.de](http://www.medela.de)**

### **Italy**

Medela Italia Srl  
Via Stelloni 39/O  
40010 Sala Bolognese  
Phone +39 051 681 52 76  
Fax +39 051 681 52 77  
e-mail [info@medela.it](mailto:info@medela.it)  
**[www.medela.it](http://www.medela.it)**

### **Netherlands/Belgium**

Medela Benelux BV  
De Steenbok 12  
5215 ME 's-Hertogenbosch  
Phone +31 73 690 40 40  
Fax +31 73 690 40 44  
e-mail [info@medela.nl](mailto:info@medela.nl)  
e-mail [info@medela.be](mailto:info@medela.be)  
**[www.medela.nl](http://www.medela.nl)**  
**[www.medela.be](http://www.medela.be)**

### **Switzerland**

Medela AG, Medizintechnik  
Lättichstrasse 4b  
6341 Baar/Switzerland  
Phone +41 (0)848 633 352  
Fax +41 (0)41 769 51 00  
e-mail [contact@medela.ch](mailto:contact@medela.ch)  
**[www.medela.ch](http://www.medela.ch)**

### **United Kingdom**

Medela UK Ltd.  
Huntsman Drive  
Northbank Industrial Park  
Irlam, Manchester M44 5EG  
Phone +44 870 950 5994  
Fax +44 870 389 2233  
e-mail [info@medela.co.uk](mailto:info@medela.co.uk)  
**[www.medela.co.uk](http://www.medela.co.uk)**